

Storing Fresenius Kabi Simplist Prefilled Syringes in Omnicell® XT Automated Dispensing Cabinets

When adding Simplist syringes to the Omnicell XT Automated Dispensing Cabinet, use empty bins or optimize existing inventory with the following steps.

1. Work with the appropriate pharmacy personnel to generate the Omnicell XT Automated Dispensing Cabinet Reports listed below and identify:
 - Empty bins
 - Unnecessary medications without active orders
 - Medications with long days unused and/or are not considered standard/emergency medications, at the pharmacy's discretion
2. Determine the desired Simplist products to be stocked and the desired PAR level
3. Use the product fit guide to determine the pockets needed to meet the PAR level for Simplist products
4. Remove the existing inventory from the designated bins and restock with the selected Simplist products

Omnicell XT Automated Dispensing Cabinet Reports

OmniCenter Report	Description
Inventory Aging	Manages space and costs by identifying items that age. Lists items that have remained unused for a specific number of days per cabinet. Each item displays the date and time of its most recent issue and total count.
Par vs. Usage	Lists item usage statistics within a date range to identify usage frequency. Evaluates par levels and reorder amounts accordingly. Items not issued or used infrequently are listed at the end of the report.
Pharmacy Drawer Configuration	Lists drawers, bins, and assigned items inside pharmacy drawer carriers. Identifies available space or like items in the same drawer.
Stocked Items Without Active Med Orders	Lists pharmacy items that have no active medication orders, to identify those that may be removed from a specific area.

Simplist® MicroVault® Storage



Maximum Bin Capacity for Simplist Prefilled Syringes in MicroVault tamper-evident packaging. Syringes fit in the external and internal return bin.


Omniceil® XT Automated Dispensing Cabinet - MicroVault Fit Guide

Pocket Name	1mL Microvault (syringes)
10 Bin locking (short bins)	10
10 Bin locking (long bins)	26
18 Bin locking	9
27 Bin locking (small bins)	0
27 Bin locking (large bins)	9
36 Bin locking	0
6 Bin locking double deep	80 to 100
15 Bin locking double deep	27

Omniceil® XT Anesthesia Workstation - MicroVault Fit Guide

Pocket Name	1mL Microvault (syringes)
FlexBin 3131 single dose drawers (24 long bins)	1 per long bin
FlexBin 2222 single dose drawers (48 bins)	1 per bin
FlexBin 96 single dose drawer	0
8 Bin locking drawers (short bins)	10
8 Bin locking drawers (long bins)	19
12 Bin locking drawers	10
24 Bin locking drawers	0

Simplist® Blister Packaging Storage

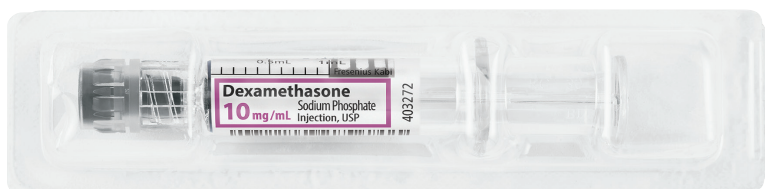

Maximum Bin Capacity for Simplist Prefilled Syringes in tamper-evident blister packaging. Syringes fit in the external and internal return bin.

Omniceil® XT Automated Dispensing Cabinet - Blister Packaging Fit Guide

Pocket Name	1mL Simplist Blister	2mL Simplist Blister	3mL Simplist Blister
10 Bin locking (short bins)	7	8	0
10 Bin locking (long bins)	18	14	10
18 Bin locking	6	6	6
27 Bin locking (small bins)	0	0	0
27 Bin locking (large bins)	6	6	0
36 Bin locking	0	0	0
6 Bin locking double deep	48	50	25
15 Bin locking double deep	22	20	4

Omniceil® XT Anesthesia Workstation - Blister Packaging Fit Guide

Pocket Name	1mL Simplist Blister	2mL Simplist Blister	3mL Simplist Blister
FlexBin 3131 single dose drawers (24 long bins)	1	1	0
FlexBin 2222 single dose drawers (48 bins)	0	0	0
FlexBin 96 single dose drawer	0	0	0
8 Bin locking drawers (short bins)	8	8	0
8 Bin locking drawers (long bins)	20	16	9
12 Bin locking drawers	8	8	0
24 Bin locking drawers	0	0	0



Maximum Bin Capacity for Simplist Prefilled Syringes in carton packaging.

Omniceil® XT Automated Dispensing Cabinet - Carton Fit Guide

Pocket Name	Carton (Adenosine 12mg/4mL, Palonosetron 0.25mg/5mL)
10 Bin locking (short bins)	2
10 Bin locking (long bins)	4
18 Bin locking	2
27 Bin locking (small bins)	0
27 Bin locking (large bins)	2
36 Bin locking	0
6 Bin locking double deep	12
15 Bin locking double deep	5
8 Bin locking drawers (short bins)	0
8 Bin locking drawers (long bins)	4